

Carefully & Safely Collect & Contain Dangerous Sharp Items

Sharps Collection Kit

Product No. SC700

The safe solution for handling, transport and disposal of high risk sharp items waste.

CONTENTS:

2 pair Nitrile Gloves
2 ea Opti Safe 700ml Sharps Containers
1 ea Ergonomic Handle: reuseable
1 ea collection 'pick up' Tong - black
2 ea PAWS Antimicrobial Wipe
2 ea Clinical Waste Bag
2 ea Cable Tie
3 x Instruction Sheets

IDC MEDICAL

13 GIBBENS ROAD WEST GOSFORD NSW
Ph: 02 4321 0299 fx: 02 4321 0288
www.idcmedical.com

SHARP ITEMS COLLECTION KIT

- Always wear protective clothing, including gloves.
- Assemble the 700ml sharps container QSopt 0.7
- Fit the QSopt 0.7 to the Ergonomic Handle
- Use the black Collection Tongs to pick up sharp items.
- Carefully deposit the sharp items into the Opti Safe sharp container
- Never deposit syringes with the needle pointing upwards -
- When full, lock the sharps container – ready for disposal
- Place the black Tongs into the Clinical Waste Bag
- Remove protective clothing & place into the clinical Waste Bag
- Clean hands with PAWS Antimicrobial Hand Wipe
- Dispose of all waste in accordance with State EPA Regulations